

# Hills Bed Card Quick Guide

24x7 helpdesk on **1300 HILLSH / 1300 445574** or  
email: [hshelpdesk@hills.com.au](mailto:hshelpdesk@hills.com.au)

The primary purpose of the Hills IP Bed Card is to display patient information.

The Bed Card Displays the following data

- The Patient Name
- The Patient record number UR
- The Doctor
- The Patient Location



At night time the screens dim to reduce the light levels in the room. At the same time as the volume drops to night level on most annunciators.

There is a software based slider on the left of the screen that allows for the screen to be made brighter. When this slider is activated, by sliding up from the tip of the arrow indicated below, the screen gets brighter and stay brighter for 10 seconds before resetting to night levels of brightness.

